



## User Information Guide

### Inflatable T-Roll

#### How to use the Inflatable T-Roll

Before unwrapping the support, please check that the product is the correct size and a good fit. As unwrapped products cannot be returned, credited or exchanged due to health and hygiene reasons. See returns policy enclosed.

The T-roll is used to control position of the body in supine lying. Pillows placed laterally along the length of the trunk lend additional support when necessary. This product is designed for short term use only.

When using the T-Roll always support the feet in order to relieve pressure at the back of the leg.

The Inflatable T-Roll should be inflated to the degree to provide a firm shape allowing approximately 12mm (1/2 inch) of depression under moderate pressure. They may be used with a Soft Knit cover available in 5 colours.

#### Note

As with all new positioning equipment, it should be gradually introduced and the use carefully monitored. If discomfort is experienced such as may interrupt sleep, discontinue use and consult your therapist.

#### Cleaning

Superficial dirt on the surface may be removed by wiping with a soft cloth moistened with hand hot water and a neutral detergent. Care should be taken when removing and replacing covers. Abrasive cleaning agents should not be used.

#### Hygiene and Disinfection

Cleaning and disinfection may be undertaken on the exterior covering. Proprietary disinfectants may be used provided the manufacturers' instructions are followed.

#### Storage

Store in a cool dry area away from direct sunlight.



### Aqua Therapy Aid **IMPORTANT**

This product has not been designed specifically as an aqua therapy aid, however some therapists have found our Inflatable T-rolls useful for stabilising position in the water, and for pre/post therapy.

- We recommend that a full risk assessment is undertaken before using our Inflatable T-Rolls as aqua therapy aids.
- Our Inflatable T-Rolls are not buoyancy or lifesaving aids and should not be used as such. Appropriate safety equipment should always be used alongside our Inflatable T-Rolls.
- Never leave a child or vulnerable person alone in the water when using an inflatable T-Roll. Full supervision is required at all times.
- Our Inflatable T-Rolls are not toys and should never be used for recreational use such as busy public swimming pools, wave machines, flumes or inflatable fun etc.

#### Storage

Ensure the Inflatable T-Roll is completely dry before folding away and storing. This will reduce the risk of weak areas and stress splits to the seams.

REVISION AUGUST 2024 LB



**C&S Seating Limited**  
**Phone** 01424 853331  
**Fax** 01424 854018

info@cands-seating.co.uk  
 www.cands-seating.co.uk

Company Reg 4718316

Offerfair Business Park  
 Ground Floor Front Office  
 12-14 Maunsell Road  
 Castleham Industrial Estate  
 St Leonards on Sea  
 East Sussex TN38 9NN

Vat No 574 3844 14